

3WAVES

User Prep Guide

Supplies required

- Alcohol wipes
- 4-5% topical lidocaine
- Disposable tissues
- Cotton tip applicator

1




- Using an isopropyl alcohol wipe, thoroughly clean a quarter-sized area behind the ears, going into the hairline if necessary and a 2"x2" square below the forehead hairline.

2




- With a cotton-tipped applicator, apply lidocaine to the cleaned area behind ear, paying attention to areas that overlap the hairline.
- Lidocaine can also be applied to the rear base (2-3mm) of the external ear.
- Wait 3-5 minutes, then wipe excess lidocaine from all sites with a disposable tissue.

3

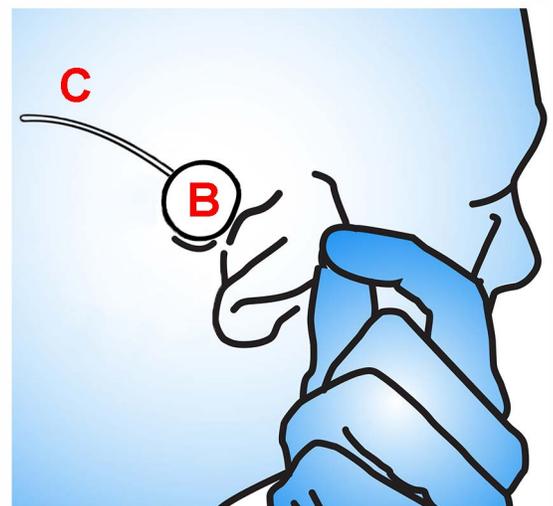

- With the external ear pulled forward, locate the bony prominence behind the ear, about halfway up (Supramastoid Crest, A).
- Place the circular electrode in the hairless area behind the ear, such that the bottom edge of the electrode rests on the highest point of the prominence (B). Ensure that the electrode does not touch the back of the ear.
- Orient the electrode leadout up and to the rear (C).
- **Important:** Reduce the amount of hair captured underneath these electrodes, as this can reduce contact efficiency.

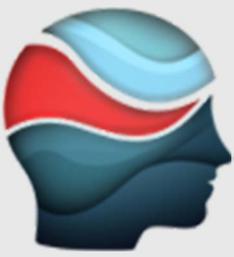


3WAVES


User Prep Guide

4

- Apply the square electrode to the center of the forehead, as high up as possible so that the top edge of the electrode is just touching the hairline.
- Orient the electrode lead facing up.

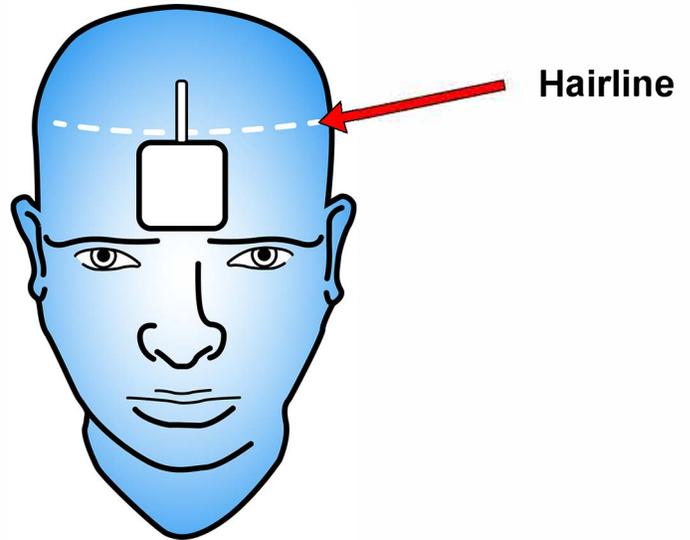

5

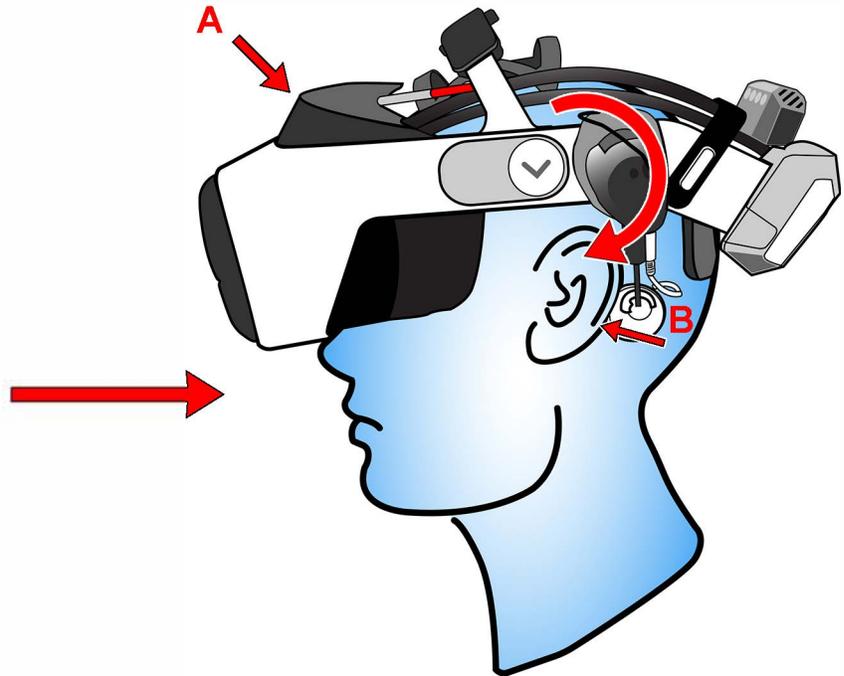

- Place the HMD onto the subject, taking care to feed the forehead lead through the top of the HMD (A).
- Rotate presser arms into place over mastoid electrodes, ensuring the foam “foot” covers the electrode and is comfortable (B).
- Plug electrodes into leads.
- To remove, disconnect electrodes and remove HMD.



This product is not intended to diagnose, treat, cure, or prevent any disease.

Persons with cochlear implants or known vestibular deficiency should consult a physician before use.